HOME VAGINAL SELF-SAMPLING HPV KIT INSTRUCTIONS

Before You Begin: SAFETY CHECK!

Read this section carefully

Review the short checklist. It helps you decide if self-testing is right for you today or if it's better to speak with a healthcare provider first.

Most people can safely use the kit—but in a few cases, it's best to wait or choose a different screening method.

You can use this self-test kit UNLESS any of the following apply:


- You are currently pregnant.
- You have given birth in the last 3 months.

- You have had a recent gynecological surgery (within the last 6 weeks).
- You are experiencing unusual vaginal bleeding or pelvic pain.
- You have had your cervix removed (e.g., hysterectomy).
- You have been advised by a healthcare provider to have your sample collected in-clinic.

If any of the above apply, please contact your healthcare provider to discuss the best screening option for you.


The test shouldn't hurt. Stop, if you feel any pain while collecting your sample, and see and ask a health care professional for help.

1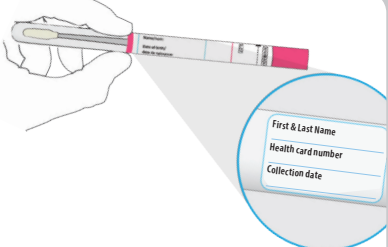

Label the tube with your name, health card number and collection date.

2

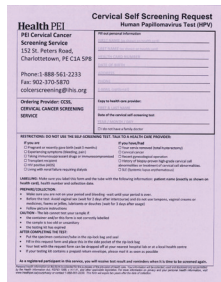
Complete the purple form. Place it in the side pocket of the plastic bag.

3


Wash your hands. Get undressed from the waist down.

4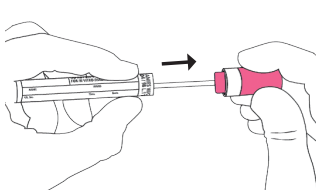


Prepare the swab: Twist and pull the cap to remove the swab. **Do not touch the soft tip!** Put the tube on a clean surface.

5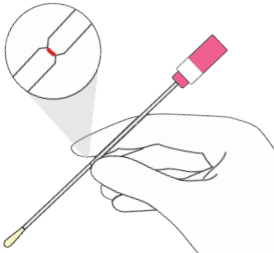

Hold the swab at the red line in one hand.


6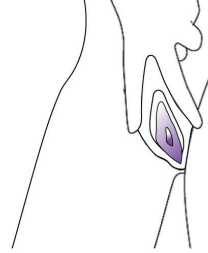

The test shouldn't hurt. Stop, if you feel pain. Stand or sit with your legs apart. Use your other hand to hold back the folds of skin.


7

Gently put the swab into your vagina, where you would normally put a tampon, until your fingers touch your outer genitals. Rotate the swab.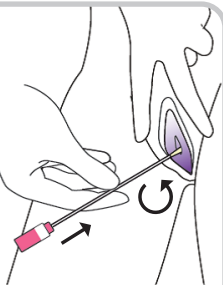


Slowly count to 20!

8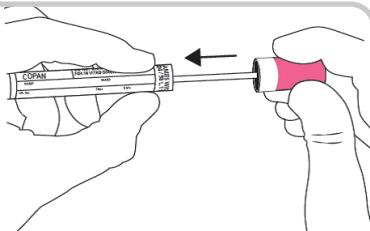

Slide the swab into the plastic tube and close firmly.

9

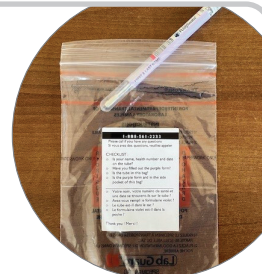
Place the tube inside the plastic bag and seal it.

FINAL STEPS: PREPARE PACKAGING

Put tube inside bag. Fold purple form and place it in the side-pocket of the bag.

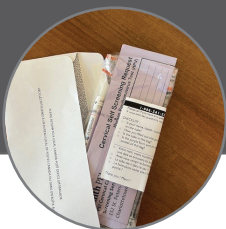Adjust plastic bag to fit the return envelope.

Seal prepaid envelope for drop off or mail to lab - free of charge.

10

Option 1: Return sample to a doctor's office, or a hospital lab.

Option 2: Mail via Canada Post to Queen Elizabeth Hospital Lab, Charlottetown.

SCAN

How to take your own sample

Video credits: ScreeningBC, Cervix Self-Screening: Instructions, 2024, via www.youtube.com

Important: To Make Sure Your Test Can Be Processed, please double-check before returning:

- Name, date of birth, health card number are correctly filled out on both the form and the tube. Must match your health card exactly.
- Date of collection on the form and tube.
- For best results return as soon as possible, test can be processed up to 21 days.
- Check expiry date on the tube, if is expired, the lab cannot process the test.

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